

Department of Public Safety - Alcohol Testing

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST R. Rosado #557 AGENCY La Paz County SO
DATE 01-23-15 TIME 08:22
INTOXILYZER SERIAL # 80-033861 (APAZSO) LOCATION APCSO BOATHOUSE

- (/) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34° C ± 0.2° C
- (/) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (/) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (/) 4. Air blank completed.
- (/) 5. Calibration check completed. Test results 0. 099 AC.
- (/) 6. Air blank completed.
- (/) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE R. Rosado #557

DPS Form Exh G-2 (Rev 05-01)

Historical Note

New Exhibit G-2 made by final rulemaking at 12 A.A.R. 1916, effective 9:00 a.m., May 18, 2006 (Supp. 06-2).

