

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R-13-10-104 (A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DEPUTY JOHN M. GOMEZ #220 AGENCY LA PAZ COUNTY SHERIFF'S OFFICE

DATE 12 / 08 / 2011

TIME 14:03 Hrs

INTOXILYZER SERIAL # 80 - 000934

LOCATION LPCSO JAIL

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.

OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34 C 0.2 C

(✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(✓) 4. Air blank completed.

(✓) 5. Calibration check completed. Test results 0. 0.099 AC.

(✓) 6. Air blank completed.

(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____


 #220

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R-13-10-104 (A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DEPUTY JOHN M. GOMEZ #220 AGENCY LA PAZ COUNTY SHERIFF'S OFFICE

DATE 12 / 08 / 2011 TIME 14:07 Hrs

INTOXILYZER SERIAL # 80 - 000934 LOCATION LPCSO JAIL

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

(✓) 1. Clock time check.

(✓) 2. Date check.

OPERATIONAL TESTS

(✓) 1. Alcohol-free subject test result 0.000 AC.

(✓) 2. Error recognition logic system functioning.

Not a Successfully Completed Test Sequence printed

(✓) 3. Proper sample recognition system.

Not a Successfully Completed Test Sequence printed

Deficient sample printed.

(✓) 4. Standard Calibration Check standard 0.100 AC Result 0.099 AC.

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE _____

INTOXILYZER 8000
Location: LPCSO JAIL
Serial Number: 80-000934
Core Version: 8105.48
12/08/2011

02:07:16

Standard Lot#: 719919
Last Changed By: J. GOMEZ #220

QAS: J. GOMEZ #220
LA PAZ CSO

Operator: J. GOMEZ #220
LA PAZ CSO

Subject: SQAP,TEST
DOB: 01/02/1975
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	02:08:26
Diagnostic Test	Pass	02:09:01
Air Blank	0.000	02:09:28
0.100 Cal Check	0.099	02:09:49
Air Blank	0.000	02:10:19
Subject Test	0.000	02:10:37
Air Blank	ABT*	02:10:43
Air Blank	IPS**	02:10:51

*Sequence Aborted
**Improper Sample

JMG #220

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: LPCSO JAIL
Serial Number: 80-000934
Core Version: 8105.48
12/08/2011

02:10:56

Standard Lot#: 719919
Last Changed By: J. GOMEZ #220

QAS: J. GOMEZ #220
LA PAZ CSO

Operator: J. GOMEZ #220
LA PAZ CSO

Subject: SQAP,TEST
DOB: 01/02/1975
Sex: M Weight: 175

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	02:11:54
Diagnostic Test	Pass	02:12:29
Air Blank	0.000	02:12:56
0.100 Cal Check	0.099	02:13:17
Air Blank	0.000	02:13:46
Subject Test	0.046*	02:16:54
Air Blank	0.000	02:17:25
Air Blank	0.000	02:17:53
Subject Test	XXX**	02:18:14
Air Blank	RFI***	02:18:22

*Deficient Sample

**Invalid Sample

Begin new deprivation period

***RFI Detect

JMG #220

Not a Successfully Completed Test Sequence

EXHIBIT G-2
THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY A.A.C. R-13-10-104 (A)
ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST DEPUTY JOHN M. GOMEZ #220 **AGENCY** LA PAZ COUNTY SHERIFF'S OFFICE

DATE 12 / 08 / 2011 **TIME** 14:19 Hrs

INTOXILYZER SERIAL # 80 - 000934 **LOCATION** LPCSO JAIL

(✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0.100 AC.

OR

Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34 C 0.2 C

(✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"

(✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.

(✓) 4. Air blank completed.

(✓) 5. Calibration check completed. Test results 0.099 AC.

(✓) 6. Air blank completed.

(✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE _____

 220

INTOXILYZER 8000
Location: LPCSO JAIL
Serial Number: 80-000934
Core Version: 8105.48
12/08/2011

02:19:51

Standard Lot#: 719919
Last Changed By: J. GOMEZ #220

QAS: J. GOMEZ #220
LA PAZ CSO

Operator: J. GOMEZ #220
LA PAZ CSO

Test	g/210L	Time
Air Blank	0.000	02:20:21
0.100 Cal Check	0.099	02:20:42
Air Blank	0.000	02:21:11

Preventative Maintenance
Performed -- Timer Reset

J. Gomez #220